

## NANO CAVITATION AND WELLA SHAPE TECHNOLOGY COMBINED IN ONE SINGLE DEVICE!


NANO CAVITATION, WELLA SHAPE RADIOFREQUENCY, INFRARED, VACUUM AND ROLES MASSAGE

- ✓ Cellulite Treatment
- ✓ Local Slimming
- ✓ Body Shaping
- ✓ Facelift
- ✓ Face Shaping




## VACUUM + NANO CAVITATION

Vacuum Assisted NANO Cavitation. The patented technology of the CROSS brand is 2 times stronger and 10 times more harmless than standard cavitation technologies. Cavitation; Non-surgical, local weight loss and cellulite treatment. Cavitation ultrasound (Low frequency sound waves) is used. These sound waves applied to the outer surface of the skin cause sudden and high pressure changes in the fluid of the fat tissue. The papillae that form together, however, first expand, then burst. This effect is called cavitation; It destroys the walls and the structure of the fat cells. Fat cells in this tissue and the fatty acids that are released are broken up and then the lymphatic pathways trying to remove it. NANO CAVITATION uses vacuum technology and cross-over technology unlike previous cavitation systems, in which area of application is vacuumed, the sound waves are focused only on the oil area so that the energy is only used in problem areas. At this point, the energy remains in the problematic area by not going to any internal organs or undesired area at all.

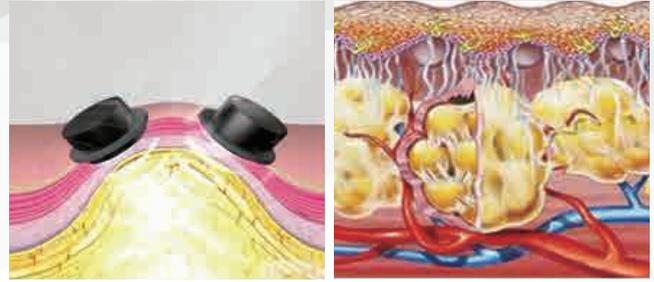

Furthermore, in the cross shot technology, since the application is made with the double head to same area at the same time, the highest result is obtained in all the fat cells in that area.

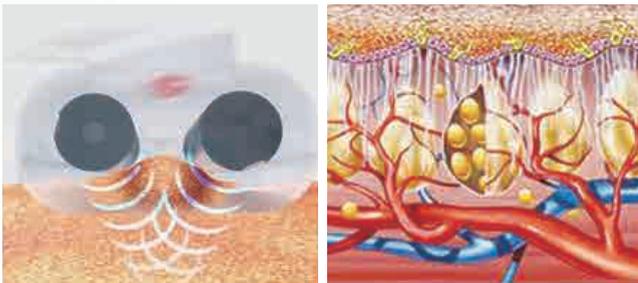

## WELLASHAPE RADIOFREQUENCY, INFRARED, VACUUM, ROLLER MASSAGE

### WELLA SHAPE

This is a local slimming device which combines four systems at the same time such as radiofrequency, infrared, vacuum and roller massage that helps to get slim locally and to treat cellulite.

### RADIOFREQUENCY

Radiofrequency is a thermoregulatory attenuation method. Radio waves activate the water molecules and ions (sodium-potassium-calcium-magnesium) in the body. Depending on their activation, the energy is released and the zone of application warms up.

The temperature of the applied region warms up to 41-43 degrees. (Normally, the body temperature is 36-37 degrees). When the skin temperature reaches this point, the temperature in the subcutaneous area corresponds to 50-55 degrees.

This heat does not cause injuries nor any harmful influence to the body. However, this high temperature reaches the fat cells between 2-22 mm depth and allows the membranes of these cells to be affected. In this way, the fat cells melt. This technology allows to reach fat cells with very high water level. Thus, fat tissue can be melt. Due to increased circulation, soluble oils are absorbed removed from the body by blood or lymph.

The old and torn collagen tissue breaks down and evaporates. Instead, the body starts producing new collagen. The tissue which provides body tightness is called connective tissue. The cells, that make up this tissue are fibroblast cells. The resulting heat accelerates fibroblast activation. The proliferation of fats and collagen, the renewal of the connective tissue ensures local slimming, the treatment of cellulite and elimination of wrinkles.

### INFRARED RAY

The infrared beam heats the tissue to a depth of 2-10 mm. When used in conjunction with the radiofrequency, it helps to increase the heat and supports radiofrequency processing.

### VACUUM

Vacuum therapy with its reverse resistance separates the oils and applied pressure breaks their resistance. In this way, there happens decrease and relaxation of fat tissue with the following acceleration of it from the body. It especially effective in cellulite treatments. It shapes body lines, helps tighten, repairs sagging and accelerates blood circulation and provides skin beauty, supports the disposal of broken fats and helps to lose weight.

### ROLLER MECHANICAL MASSAGE

Mechanical massage provides lymph drainage (Dermal and Hipodermal). It helps to relief tension points and spasms, eliminate edemas and lactic acid that accumulates in muscles after fatigue. Rhythmic massage accelerates blood circulation, reduces the aging lines of the skin and gives skin elasticity.

It is one of the fastest growing methods used especially for local slimming. Thanks to the applied massage, the blood flow is accelerated, the veins are revealed and the body becomes more elastic. Massage can be applied to the whole body, but it is used more effectively in the hip, upper leg, arm and buttocks areas aimed to treat cellulite.



### **BODY VACUUM NANO CAVITATION HEAD**

Thanks to the unrivaled head produced in accordance with the patented technology, you can get great results in local slimming and cellulite treatment. The head has LCD display and control key system.

Due to this, you can set start up and stop off buttons and set up the applications both on the device and on the head screen. This head is suitable for abdominal and hips applications.



### **BODY RADIOFREQUENCY, INFRARED, VACUUM, ROLLS MASSAGE BIG HEAD**

It is a special very big head which combines four systems such as radiofrequency, infrared ray, vacuum and roller massage and helps provide local slimming and treat cellulite. It is suitable for large areas of our bodies such as abdomen, hips, and back. The head has LCD display and control key system. Due to this, you can set start up and stop off buttons and set up the applications both on the device and on the head screen.

Special filter system of this device inspipe the fact that free radicals and dead cells are being absorbed in the filters allow to avoid damage of device and its head. There is also no need to wear a special costume.



### **BODY RADIOFREQUENCY, INFRARED, VACUUM, ROLLS MASSAGE SMALL HEAD**

It is a special very small head which combines four systems such as radiofrequency, infrared ray, vacuum and roller massage and helps to provide local slimming and treat cellulite. It is suitable for narrow areas such as arm, shoulder, lower leg zones. The head has LCD display and control key system. Due to this, you can set start up and stop off buttons and set up the applications both on the device and on the head screen. Special filter system of this device inspipe the fact that free radicals and dead cells are being absorbed in the filters allow to avoid damage of device and its head. There is also no need to wear a special costume.




### **FACE RF + VACUUM HEAD**

It is a special head which has 3 separate head ends that combine radiofrequency and vacuum system together. It is used for facelift, wrinkle treatment, eyelid tightening and eye wrinkle treatment. According to the area application, head ends can be easily replaced.



<b>VACUUM</b>	10 Kpa - 90 Kpa (5.7 cmHG – 68,4 cmHG)
<b>VACUUM RANGE</b>	0,5 S – 7,5 S
<b>ROLLER SPEED</b>	0-36 rpm
<b>ROLLER OPERATION MODE</b>	2 Different Modes
<b>NANO CAVITATION POWER</b>	2 x 60 W
<b>NANO CAVITATION FREQUENCY</b>	2 X 40 Khz
<b>RADIO FREQUENCY POWER</b>	up to 60 W
<b>INFRARED LED POWER</b>	up to 30 W
<b>LASER WAVE SIZE</b>	940 nm
<b>VACUUM</b>	Fitful

## CLINICAL RESULTS

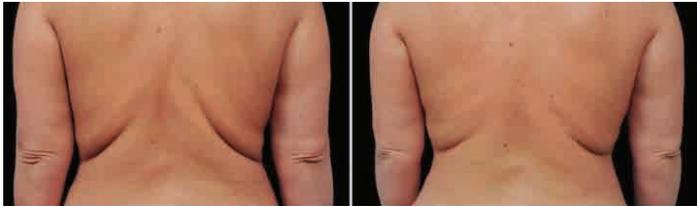

Before

After



Before

After



Before

After



Before

After

